


Use the Provation® Apex Desktop application to capture images previously stored on an Olympus processor.


Walkthrough: Import Images into Provation Apex from Olympus Processors

To transfer stored images from an Olympus CV-190 processor, navigate to the image(s) on the processor, then capture them using the Image Capture screen in Provation Apex.


1. Press the **Menu** key on the Olympus keyboard.
2. Use the **arrow** and **ENTER** buttons on the Olympus keyboard to select **View Saved Images**.

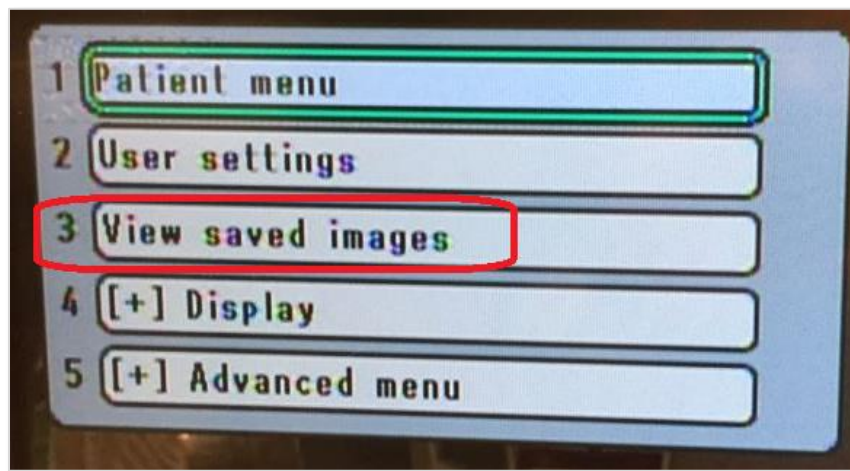


3. On the View Buffer Images screen, arrow down to **Review Image (S)**, then use the keyboard's right arrow to navigate to patients. When the appropriate patient is selected, press **ENTER** to view the images.

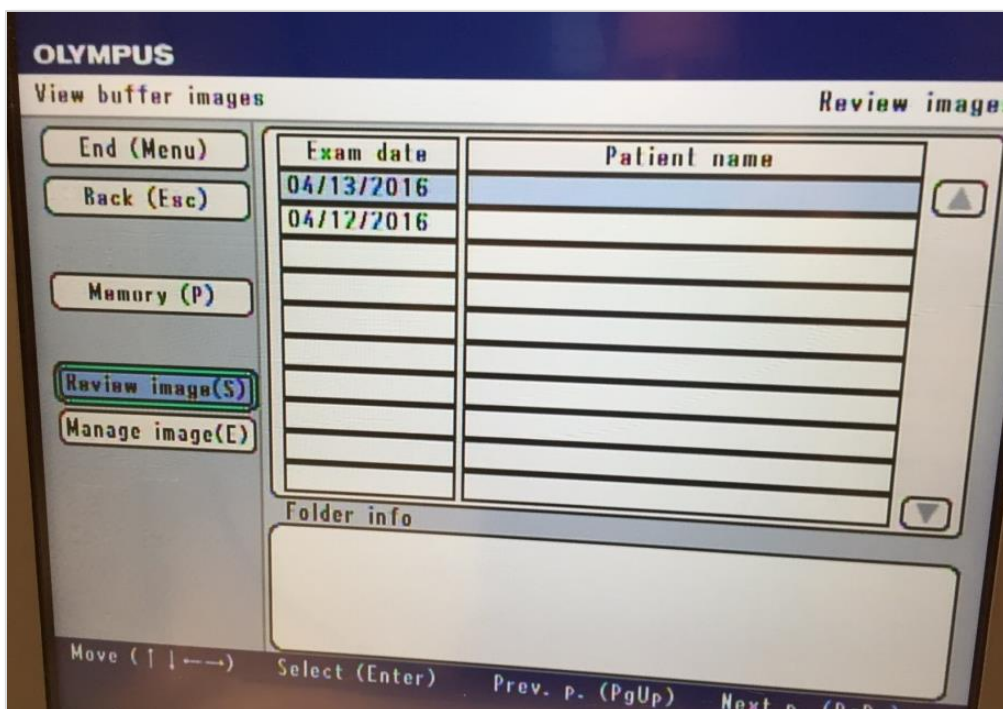


4. Use the arrows to navigate to the **View (V)** button, then press **ENTER**.

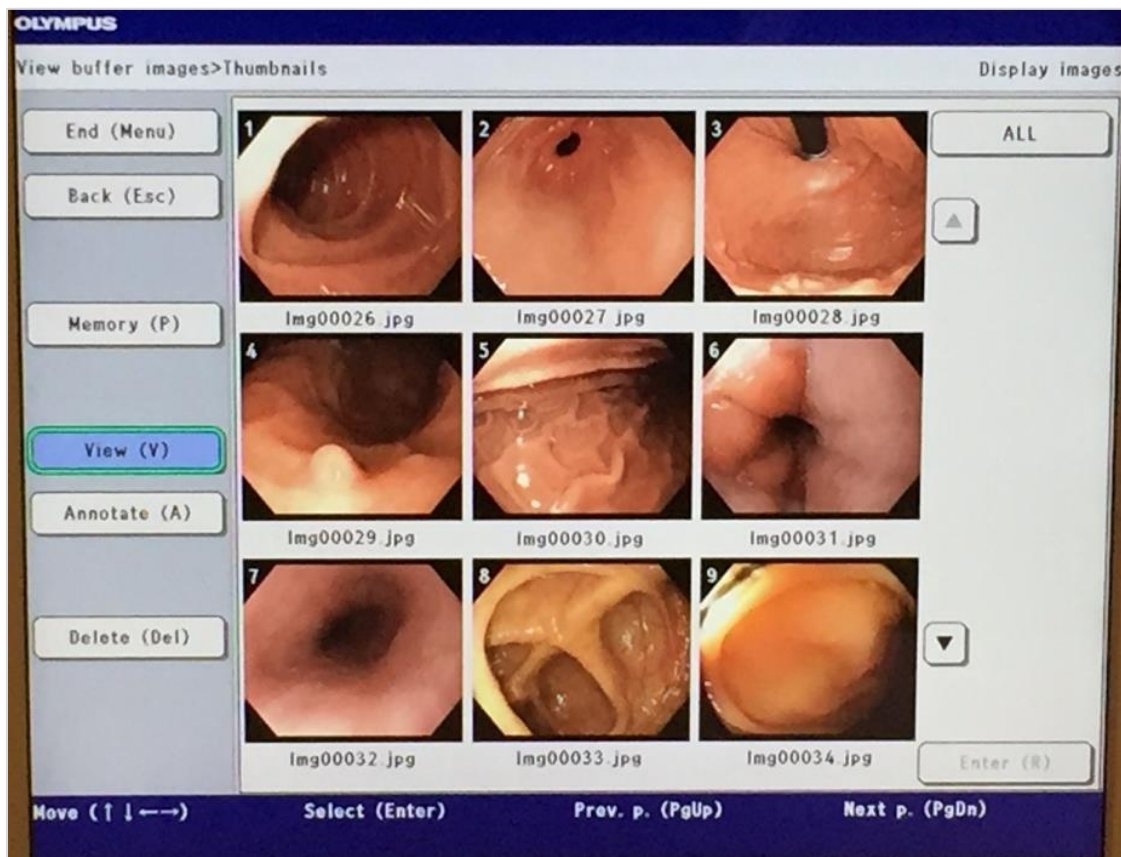


5. The **ALL** button in the upper right corner has now activated.

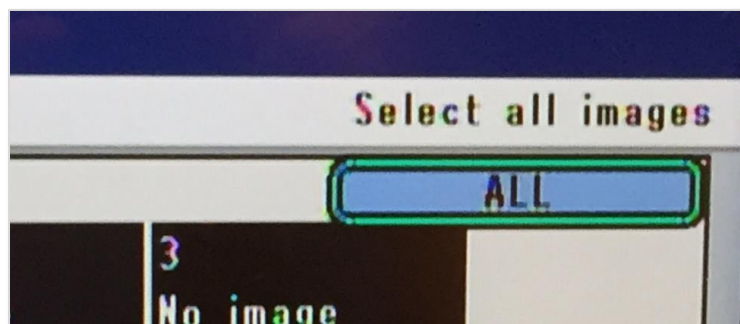

Press **ENTER** to queue that case's images. Queued images are marked with a purple rectangle.

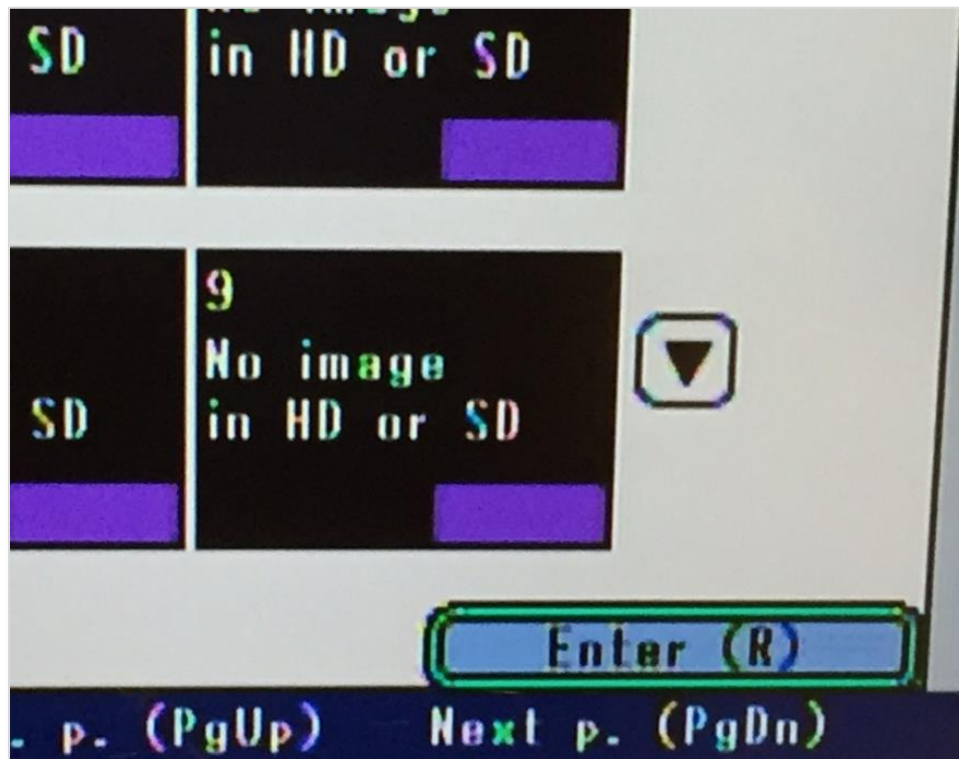


Press **ENTER** to cycle through the queued images one by one. Each time ENTER is pressed, one image displays and may be captured within Provation Apex. If there are more than nine images, press the **black triangle** button on the Olympus screen to page down to view them.

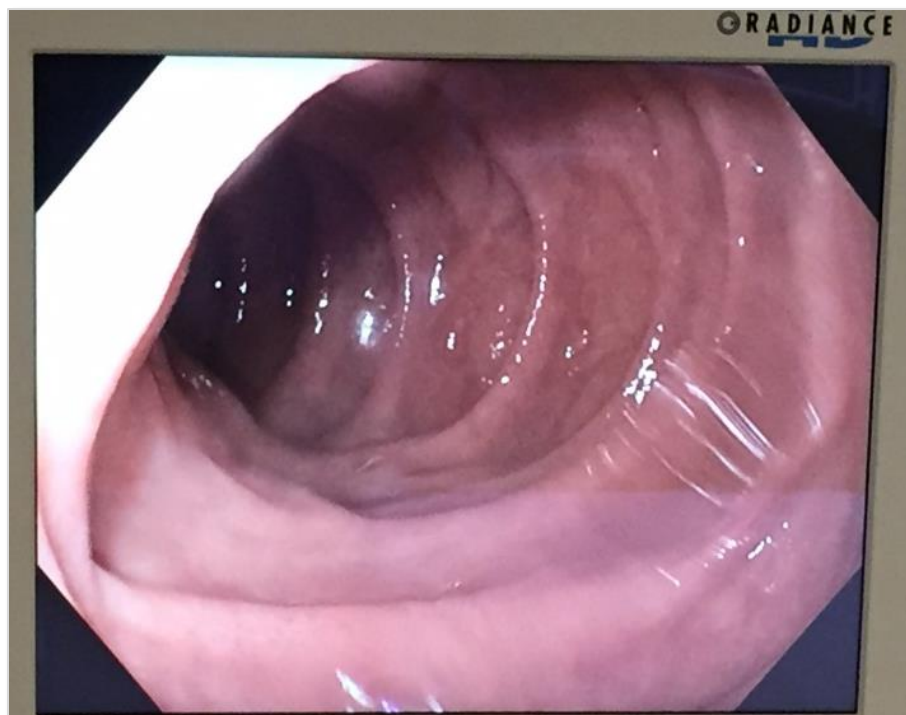

To access the image capture screen in Provation Apex:

1. At the bottom of a newly-scheduled case's Case Information window, select **Capture Images**.

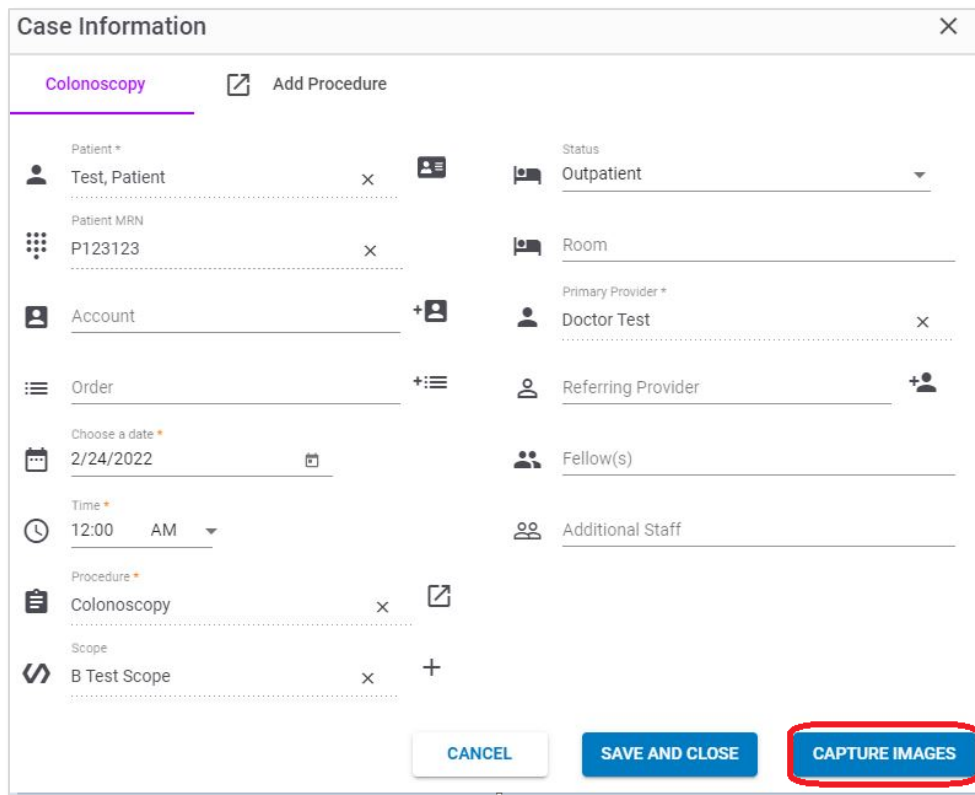

Case Information [X]

Colonoscopy [Add Procedure]

Patient *
Test, Patient [x] [icon]

Patient MRN
P123123 [x]

Account [x] [icon]

Order [x] [icon]

Choose a date *
2/24/2022 [calendar icon]

Time *
12:00 AM [dropdown]

Procedure *
Colonoscopy [x] [icon]

Scope
B Test Scope [x] [icon]

Status
Outpatient [dropdown]

Room

Primary Provider *
Doctor Test [x]


Referring Provider [x] [icon]

Fellow(s)

Additional Staff

[CANCEL] [SAVE AND CLOSE] [CAPTURE IMAGES]

Or, from within an existing case, select the **Capture Images** icon on the top right toolbar:

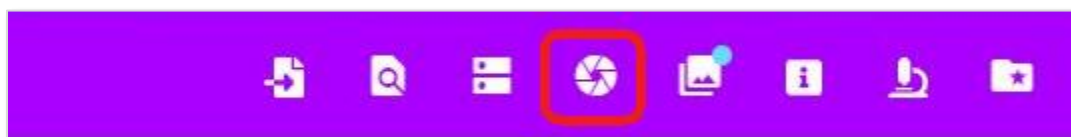

2. The video feed appears in the center of the screen, and displays the images from the Olympus processor. Press the "P" key on the computer keyboard to capture the image in Provation Apex, or select the **capture** icon in the image capture window.

